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Exerto Ósseo Baumer - Orgânico (GenOx Org) is a resorbable implantable medical product composed of lyophilized demineralized bovine bone. Exerto Ósseo Baumer - Orgânico (GenOx Org) was designed to be used as an osteoconductive bone substitute for filling bone defects by dental and medical professionals.

Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented as opaque yellowish-white granules or blocks, the granules being presented in single glass bottles packed in blisters and the blocks packed in double blisters.

Exerto Ósseo Baumer - Orgânico (GenOx Org) is a product of organic composition with a high degree of purity, free from contamination with heavy metals. It is an acellular, non-cytotoxic, non-immunogenic, non-pyrogenic and biocompatible product.

Exerto Ósseo Baumer - Orgânico (GenOx Org) does not provide structural support, and there is recommended for filling in stable bone defects closed by walls, such as cavities and bone cysts, except for processes where their use is a diagnosis of uncontrolled cell growth. Electron microscopy studies show the presence of pores in the Enxerto Ósseo Baumer - Orgânico (GenOx Org), which makes cells with osteogenic potential easier to deposit and adhere to the material, and consequently, a better condition for proliferation and necessary morphological maturation. For bone matrix production.

Exerto Ósseo Baumer - Orgânico (GenOx Org) can be used alone or in combination with autogenous bone, or even associated with other bone substitute biomaterials of synthetic or natural origin (such as deproteinized bone or recombinant human bone morphogenetic protein or bovine).

Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented in two models: GenOx Org Cortical and GenOx Org Cancellous which are produced through the application of the same standard physico-chemical process for removing the mineral portion (decalcification) of young bovine bones. GenOx Org Cortical is produced through the process of descaling the cortical portion of bones of young cattle while GenOx Org Spongy is produced through the same process of descaling applied to the medullary portion of bones of young cattle.

Exerto Ósseo Baumer Orgânico (GenOx Org) is commercialized STERILE, being STERILIZED with Gamma RADIATION. Exerto Ósseo Baumer - Orgânico (GenOx Org) is a single use product, being PROHIBITED to REPROCESS.

For greater patient comfort and safety, adhesive labels containing information for product traceability (such as batch number, expiration, etc.) are available inside the final packaging.

FUNDAMENTALS OF THE PRODUCT OPERATION AND ACTION

The production process of Enxerto Ósseo Baumer - Orgânico (GenOx Org) preserves the original architecture and structural characteristics of the bone while maintaining the organic portion of the bone, allowing the final product to act as an osteoconductive element at the implant site. Exerto Ósseo Baumer - Orgânico (GenOx Org) is a biocompatible, acellular, non-cytotoxic, non-immunogenic, non-pyrogenic and high-purity bone substitute, as determined in vivo and in vitro studies. Due to its osteoconductive action and porosity, it acts as a support for deposition of new bone being reabsorbed and leaving new bone in place, allowing this product to be used in isolation or associated with other graft materials that have osteoinductive or osteogenic activity. Due to its chemical characteristics (organic nature), Exerto Ósseo Baumer - Orgânico (GenOx Org) is more quickly absorbed than totally deproteinized xenografts, being absorbed and integrated during the normal bone repair process by providing cells with osteogenic potential. Adhesion more favorable and more appropriate conditions to proliferate and finish its morphological maturation and finally produce bone matrix.

CONTENTS AND PRESENTATION

Exerto Ósseo Baumer - Orgânico (GenOx Org) is presented lyophilized and STERILE, the product being in the form of granules packaged in a glass bottle and presented in the form of blocks packaged in a double blister. The flask containing the product in granules or the blister containing the product in blocks is packed inside a blister (secondary packaging - plastic blister sealed in surgical grade paper) disposed in a cardboard box - final packaging. The box is labeled with the relevant data and information and laminated with heat shrinkable plastic. The package insert for the medical product (instructions for use) is available inside the final packaging - box.

Self-adhesive labels (five) for product traceability are available inside the final packaging. The following information and traceability items are found on the labels: Product model, technical name and trade name, Manufacturer's logo and address, Content description, Quantity and product size data, Product reference code, Production batch, Manufacturing date, Shelf life, Product registration number, Technical manager.

It is mandatory for the dentist or doctor to attach a label (label No. 1) to the patient's medical record or equivalent document, in order to comply with the current health legislation.

In order to continue the traceability cycle, the manufacturer recommends that the health professional assign the other available labels (labels nº2 to nº5) as follows:

Label Nº2: deliver to the patient

Label Nº3: fix in the tax documentation that generates collection (AIH in the case of a SUS patient, or invoice in the case of a patient assisted by the complementary health system).

Label Nº4: forward to supplier control

Label Nº5: forward to the control of the responsible (main) surgeon.

It is the responsibility of the hospital institution to ensure the identification and traceability of the medical device at the time of use through the adhesive label provided.

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Figura 2: Detalhe do produto médico Bone Graft Baumer - Orgânico (GenOx Org), em grânulos dentro do primário (vidro à esquerda) e secundário (blister selado em papel grau cirúrgico). Note that a special adhesive chemical indicator for sterilized products with gamma radiation changes the color (from yellow to yellow), red after coming into contact with the sterilizing agent. A γ symbol is printed on each indicator, in addition to the indication that red means exposed to radiation (although the English words are: Red is sterilized).

The product Enxerto Ósseo Baumer - Orgânico model GenOx Org Cortical, which is composed of bovine bone of cortical origin demineralized by a standardized process of removing the mineral portion of the bones, is presented in the form of granules or blocks. The granules are made available to the market in granulometries from 0.25 mm to 9.0 mm in doses from 0.5cc (0.25g) to 400.0cc (200.0g), with each cubic centimeter (cc) equivalent to the weight approximately 0.5 gram. The blocks are presented in three dimensions in packages containing one to eight blocks each. Below are the codes for the product presentations.

The product Baumer Bone Graft - Organic model GenOx Org Spongy, which is composed of bovine bone of spinal (spongy) origin by a standardized process of removing the mineral portion of the bones, is presented in the form of granules or blocks. The granules are made available to the market in granulometries from 0.25 mm to 9.0 mm in doses from 0.5cc (0.1g) to 400.0cc (80.0g), with each cubic centimeter (cc) equivalent to the weight approximately 0.2 gram. The blocks are presented in three dimensions in packages containing one to eight blocks each. Below are the codes for the product presentations.

Below are tables of the product Enxerto Ósseo Orgânico Baumer Model: GenOx Org Cortical presentation in granules containing the product codes for presentations by granule size, their doses in cc and weight in grams.

Enxerto Ósseo Baumer - Orgânico			
GenOx Org Cortical grânulos de 0,50 to 1,0mm			
Model	Code	Dose (cc)	Weight (Aprox. g)
GenOx Org Cortical	931.50.0,5	0,5cc	0,3g
GenOx Org Cortical	931.50.1	1,0cc	0,5g

The information regarding the use, safety, handling, transport, storage and traceability of the product Enxerto Ósseo Baumer Orgânico is presented as symbols as described in the chart below:

SÍMBOLO	MEANING
	Single-use product
	Caution, consult accompanying documents
	Handle with care.
	Maximum temperature.
	Keep dry.
	Date of manufacture.
	Valid until.
	Batch Code.
	Number of catalog.
	Sterilized by Gamma Ray.
	Do not use if package is damaged.
	CAUTION: FEDERAL LAW IN USA AND CANADA REQUIRES THIS DEVICE TO BE SOLD BY OR ON THE ORDER OF A PHARMACIAN

NOTE: The graphic symbols related to the labeling of the medical product, referenced above, meet the requirements established in ISO 15223: 2000 Medical devices – Symbols to be used with medical device labels, labelling, and information to be supplied; ISO 15223: 2000 / Amd 1: 2002; and ISO 15223: 2000 / Amd 2: 2004.

SUPPORT MATERIAL

Support materials that come with the medical device are:

- Product use instructions
- Identification label for protocol and patient

COMPOSITION

Baumer Bone Graft - Organic (GenOx Org) is a material of organic nature, product of the demineralization of cortical or medullary bovine bone by chemical and thermal processing until obtaining granules or blocks. Baumer Bone Graft - Organic (GenOx Org) is a product of an organic nature and in the monitoring of the physicochemical characteristics of Baumer Bone Graft - Organic (GenOx Org) the groups of components determined are presented in the following percentages: Fat = < 1%, Protein => 90%, Ash = < 5%, pH = 6-8. Heavy metals As = < 3ppm, Cu = < 3ppm, Cd = < 3ppm, Cr = < 3ppm, Hg = < 5ppm, Pb = < 30ppm.

INDICATIONS

Enxerto Ósseo Baumer Orgânico (GenOx Org) is indicated for use as a bone substitute of osteoconductive action for filling bone defects by dental and medical professionals. Enxerto Ósseo Baumer Orgânico (GenOx Org) was designed to be used in cavity filling procedures where there has been bone loss and which needs to be replaced by bone tissue. The product must be used in places that have three bone walls allowing physical support to the material and maintenance of the material in place. This product was not designed to be used in the areas of plastic and aesthetic surgery as a material for subcutaneous filling. This product can be used alone or associated with other bone substitute materials such as autogenous or autologous grafts. This product should not be used alone or associated with other bone substitute products in situations where there is uncontrolled cell growth (for example, cysts or tumors with suspected or diagnosed malignancy). Enxerto Ósseo Baumer Orgânico (GenOx Org) acts as an osteoconductive element functioning as biological support for cell development

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